(GB) Instructions for use	6
(ES) Instrucciones de uso	15
(BR) Instruções de uso	24
(GR) Οδηγίες χρήσης	33
(CZ) Návod k použití	42
(HU) Használati utasítás	51
(LV) Lietošanas instrukcija	60
(LT) Naudojimo instrukcija	69
(PL) Instrukcja obsługi	78
(HR) Upute za uporabu	87
(RO) Instrucțiuni de utilizare	96
(TR) Kullanma talimatı	105
(RU) Инструкция по применению	114
(AE) تعليمات الاستخدام	123
(CN) 使用說明	130
(SK) Návod na použitie	138
(BG) Инструкции за употреба	147
(UA) Інструкція із застосування	156
(EE) Udhëzime përdorimi	165
(SI) Navodila za uporabo	174
(RS) Uputstva za upotrebu	183
(MK) Упатство за употреба	192

Pasty
Pastoso
Pastoso
ζυμώδες
Pastovité
Pasztaszerű anyag
Pastveida
Pastos pavidalo
W postaci pasty
U obliku paste
Pastã
Macun gibi
Пастообразное
مادة عجينية
膏狀
Pastovítý
пастьозно
Пастоподібний
Pastakujuline
Pastozen
U obliku paste
Тестесто

Explanation of symbols on labelling

Medical device

Reorder no.

Manufacturer

Batch code

Use-by date

Date of manufacture

Unique Device Identification

Consult instructions for use

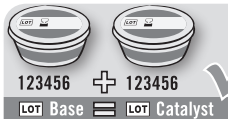
Storage temperature


Keep dry

Contains hazardous substances

A


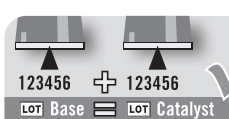
B


Description of applicational / packaging icons

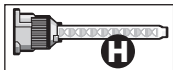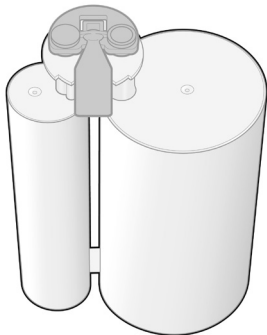- A** = Only use pastes with same LOT No
- B** = Icon "Use of the special program at the Dynamix® speed Mixing Unit possible"
- C** = Mixing ratio
- D** = Mixing time
- E** = Extraoral working time
- F** = Intraoral working time
- G** = Time in mouth
- H** = Static mixing tip
- K** = Intraoral tip

Dynamix Cartridge, 380 ml

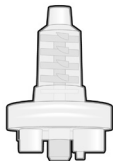
Dynamix Mixing Tip, yellow

Dynamix Fixation Ring

Cartridge System, 50 ml

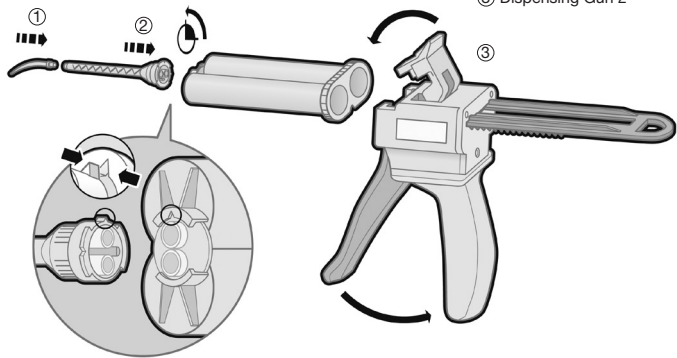- ① Intra Oral Tip
- ② Mixing Tip
- ③ Dispensing Gun 2

Medical Device – for use by dental health care professionals only.

Intended purpose

Addition curing polyvinyl siloxane for impressions, for direct application.

Indications

Indicated for all impression techniques.

Contraindications

The use of this product is contraindicated in case of known or suspected allergies against components of this product.

Patient target group

This dental material is suitable for patients requiring dental treatment for the mentioned indications with consideration of the contraindications. For pregnant and nursing women, the treatment risks must be weighed carefully against the benefits taking into consideration the unborn child or infant.

Intended user

For use by dental health care professionals only.

Clinical benefit

Kulzer medical devices ensure the rehabilitation of oral functions such as chewing, speaking and aesthetics. They stabilize the remaining dentition and/or the alveolar ridge restoratively or prosthetically or they aid in the rehabilitation process.

Potential side effects

Hypersensitivities to the product or its components cannot be excluded in individual cases. In case of suspicion please contact the manufacturer for ingredients information.

Composition

Elastomeric, silicone-based precision impression material: polyvinyl siloxane, addition curing

Material

Pack

Variotime Easy Putty

Tub, 1:1, 2 x 300 ml

Variotime Dynamix Putty

Cartridge, 5:1, 380 ml

Variotime Heavy Tray

Cartridge, 1:1, 50 ml

Variotime Dynamix Heavy Tray

Cartridge, 5:1, 380 ml

Variotime Monophase

Cartridge, 1:1, 50 ml

Variotime Dynamix Monophase

Cartridge, 5:1, 380 ml

Variotime Medium Flow

Cartridge, 1:1, 50 ml

Variotime Light Flow

Cartridge, 1:1, 50 ml

Variotime Extra Light Flow

Cartridge, 1:1, 50 ml

Variotime unlimited

Cartridge, 1:1, 50 ml

Variotime Dynamix unlimited

Cartridge, 5:1, 380 ml

Product application

The Variotime range is an optimally matched system and its different types of material can be combined for all impression techniques to suit specific requirements.

Not to be used in conjunction with other silicone materials. Note for implants: Follow the implant manufacturer's instructions regarding the impression technique and procedure steps.

Variotime	EN ISO 4823	Product colour		Mixing ratio	Mixing time	Extraoral working time* (23°C (73°F))	Intraoral working time (35°C (95°F))	Minimum time in mouth	Part of packaging material	
		Base	Catalyst						Mixing Tips	Intraoral Tips
Variotime Easy Putty	Type 0 putty	violet	light grey	1:1	30	2:30	–	2:30	n.a.	n.a.
Variotime Dynamix Putty	Type 0 putty	violet	light grey	5:1	auto	2:30**	–	2:30	dynamic yellow 40004353	n.a.
Variotime Heavy Tray	Type 1 heavy bodied	violet	light grey	1:1	auto	2:30	–	2:30	green 66001989	n.a.
Variotime Dynamix Heavy Tray	Type 1 heavy bodied	violet	light grey	5:1	auto	2:30**	–	2:30	dynamic yellow 40004353	n.a.
Variotime Monophase	Type 2 medium bodied	purple	light grey	1:1	auto	2:30	1:00	2:30	green 66001989	white 65687514
Variotime Dynamix Monophase	Type 2 medium bodied	purple	light grey	5:1	auto	2:30**	1:00	2:30	dynamic yellow 40004353	n.a.
Variotime Medium Flow	Type 2 medium bodied	light green	light grey	1:1	auto	2:30	1:00	2:30	yellow 66001988	yellow 66000782
Variotime Light Flow	Type 3 light bodied	light green	light grey	1:1	auto	2:30	1:00	2:30	yellow 66001988	yellow 66000782
Variotime Extra Light Flow	Type 3 light bodied	green	light grey	1:1	auto	2:30	1:00	2:30	yellow 66001988	yellow 66000782
Variotime unlimited	Type 2 medium bodied	white	dark grey	1:1	auto	2:30	1:00	2:30	green 66001989	white 65687514
Variotime Dynamix unlimited	Type 2 medium bodied	white	dark grey	5:1	auto	2:30**	1:00	2:30	dynamic yellow 40004353	n.a.

*At room temperature of 23°C (73°F), 50% rel. humidity. Higher temperatures shorten, lower temperatures prolong these times. ** Measured using the Dynamix mixer from Kulzer.

Tubs must be carefully sealed after use. Do not inter-change caps/lids/spoons. Dispense and discard a small amount of material from a 50ml cartridge before attaching a mixing tip. After use, leave the mixing tip on the cartridge to keep it closed. Immediately before the next use, remove the mixing tip, dispense and discard a small amount of material. Attach a new mixing tip, and again dispense and discard a small amount of material. Mixing tips and intraoral tips are single-use products. To prevent cross-contamination between patients, mixing tips and intraoral tips must be discarded after each use.

In all addition-curing silicone impression materials, the setting process may be affected by contact with products containing active Sulfur or Nitrogen, or by fresh composite fillings or superstructures. These products include latex gloves, Polysulfide impression materials, retraction solutions, etc. Variotime contains surfactants for hydrophilation, which in some patients may cause a superficial anesthetic effect.

Surplus material on the impression tray or in the patient's mouth should be avoided as this may result in gagging, swallowing or aspiration. To further reduce these risks during impression taking, the patient should be positioned as upright as possible.

Do not leave any residual material in the sulcus or oral cavity, as this can lead to abscess formation.

If there are substantial undercuts, highly divergent teeth, or other critical conditions in the impression area, appropriate blocking out must be performed when using Variotime impression materials.

Model fabrication

The impression can be cast 30 minutes after removal from the mouth – there are no other time constraints. The impression is stable in long-term storage.

Recommended model material

All standard model materials can be used.

We recommend:

Class III plaster, MOLDANO®

Class IV plaster, MOLDASTONE®/MOLDASYNT®

Disinfection

Impressions can be treated with an aqueous sodium hypochlorite solution (5.25%, for up to 10 minutes*) or with other disinfectant solutions suitable for silicones. When doing so, please observe the disinfection/processing instructions issued by the relevant manufacturers.

* *Gelson Luis Adabo, DDS, PhD, Elaine Zanarotti, Renata Garcia Fonseca, and Carlos Alberto dos Santos Cruz, DDS, PhD – Effect of disinfectant agents on dimensional stability of elastomeric impression materials. J Prosthet Dent, 81, 621-4, 1999*

Adhesion

We recommend using our Universal Adhesive.

Dynamix System, 380 ml – step by step

Fig. 1
Remove cap
Insert cartridge
into the mixer

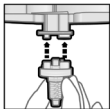
Fig. 2
Attach Dynamix
Mixing Tip

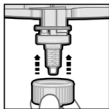
Fig. 3
Position Dynamix
fixation ring

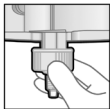
Fig. 4.1
Attach Dynamix
fixation ring
Slide over
recesses

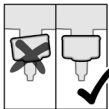
Fig. 4.2

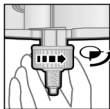
Fig. 5
Lock by giving
fixation ring
a ¼ turn
clockwise

Inserting the Dynamix cartridge

For information on how to handle the mixer being used please refer to the detailed instructions issued by the relevant manufacturer.

1. Hold the tab on the cap of the outlets and bend it upwards through 90° without applying any excessive force
2. Take the tab between your thumb and index finger whilst you hold it securely with your other hand and remove the cap from the cartridge completely (Fig. 1)
3. Place one of the enclosed Dynamix Mixing Tips over the outlets of the cartridge (Fig. 2)
4. If it proves difficult to attach the mixing tip, check to make sure that the central internal hexagon of the mixing tip is properly aligned with the hexagon of the drive spindle
5. When the mixing tip is in the correct position (Fig. 3), push the Dynamix fixation ring over the mixing tip until it stops and turn it clockwise (1/4 turn) until it is securely locked in place (Figs. 4.1, 4.2 and 5)

6. When impression material has been mixed, transfer it to a separate intraoral impression syringe, e.g. Impression Jet (Variotime Dynamix Monophase only) or a suitable impression tray
7. When the material has been dispensed, retract the mixer plungers from the cartridge in order to avoid continuous pressure and thus prevent excess material from continuing to emerge from the mixing tip

Notes about safe use of the Dynamix System

Dynamix cartridges are heavy and can be damaged if they are dropped.

Damaged cartridges must no longer be used.

Only use Dynamix cartridges in mixers with a closed protective flap.

Before using for the first time, adjust the levels of the two material components. Dispense approx. 2–3 cm material and discard this amount before starting to fill the tray. When the cartridge is used in different mixing devices, 2–3 cm of material has to be discarded after each device change. Each time before you use the material check the outlet channels of the cartridge for blockages.

At the end of a working day or if the mixing device is not going to be used for a lengthy period, remove the cartridge from the device and store it in a horizontal position with the outlets closed off by the mixing tip or cap.

After use, leave the mixing tip on the cartridge to keep it closed.

Immediately before the next use, remove the mixing tip and attach a new one.

To do so, detach the Dynamix fixation ring and use it again for the new mixing tip attached.

Only use Dynamix System components for dispensing and mixing the impression material in Dynamix cartridges.

Check the system components for physical damage on a regular basis and only use components which are intact.

Storage conditions

Do not store above 25°C (77°F). Store in a dry place. Do not use after expiry date.

Warnings and safety instructions

Polyvinyl siloxanes are chemically resistant. Material which has not set can soil clothing. **Wear protective gloves according to EN 420, category III, with protection against chemical risks according to EN 374, type C.** Avoid contact with skin and eyes. May cause skin and eye irritation. In case of contact with skin, immediately wash with plenty of water and soap. In case of contact with eyes, rinse with water for several minutes. If symptoms persist, consult an eye specialist. Do not swallow or ingest. If health problems arise after swallowing impression material, immediately contact a doctor. Intestinal blockage may arise in rare cases. May damage the unborn child.

General information

Please pay attention to the following safety instructions and all processing instructions in other sections. This product must be used according to the instruction for use only. Any use not complying with these instructions is at the discretion and sole responsibility of the dental professional itself.

Application notes

Visually inspect for damage before use. Damaged products must not be used. Variotime impression materials should be at room temperature 23°C (73°F) for processing.

Disposal information

Recommendation: Dispose of in accordance with official regulations. Do not dispose of contents or partially emptied packaging in the household waste or allow them to enter the sewage system. European Waste Catalogue: 180106 Chemicals consisting of or containing hazardous substances.

Complaint notice

Please quote batch number and article number or UDI in all correspondence about the product. Article number **REF** and batch number **LOT** or UDI: See note on packaging.

According to EU Medical Device Regulation users/ patients are obliged to report serious incidents with a medical device to the manufacturer and to the competent authority of the country, the incident occurred.

Dated: 2024-12

Description of applicational / packaging icons

A = Only use pastes with same LOT No

B = Icon "Use of the special program at the Dynamix® *speed* Mixing Unit possible"

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Kulzer GmbH