# Enhance®

Finishing System

**INSTRUCTIONS FOR USE** .....

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**MODE D'EMPLOI** .....

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Finiersystem

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Sistema de terminado

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Sistema de acabamento

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Fineersysteem

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Putssystem

**BRUKSANVISNING** .....

Pudse- og poleringssystem

**BRUGSANVISNING** .....

Pusse- og poleringssystem

**BRUKSANVISNING** .....

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**KÄYTTÖOPAS** .....

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Apdailos sistema

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**LIETOŠANAS INSTRUKCIJA** .....

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**KASUTUSJUHEND** .....

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Systém na konečné úpravy

**NÁVOD NA POUŽITIE** .....

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Sisteme de finisare

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Sistemi za završnu obradu

**UPUTSTVA ZA UPOTREBU** .....

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# Enhance<sup>®</sup> PoGo<sup>®</sup> Polishing Systems

## INSTRUCTIONS FOR USE - ENGLISH

**Caution:** This is a medical device. For dental use only by dental professionals.  
**USA:** Rx only.

### 1. PRODUCT DESCRIPTION

The Enhance<sup>®</sup> Finishers are pre-mounted, single use, aluminum oxide impregnated, cured urethane dimethacrylate resin finishers designed for preparing composite surfaces for their final polish. Enhance<sup>®</sup> PoGo<sup>®</sup> Polishers are pre-mounted, single-use diamond impregnated cured urethane dimethacrylate resin polishing devices designed for use in the final polishing of all accessible composite resin and compomer restorations. When used with a conventional (low-speed) contra-angle latch handpiece compliant with ISO 14457:2017, Enhance<sup>®</sup> Finishers will yield a smooth, contoured surface with pre-polished surface luster on composites and compomers. For optimal surface polish, follow the use of Enhance<sup>®</sup> Finishers with Enhance<sup>®</sup> PoGo<sup>®</sup> Polishers as outlined below, or Prisma<sup>®</sup> Gloss<sup>™</sup> Composite Polishing Pastes.

Shank dimensions are compliant with ISO 1797:2017 Dentistry — Shanks for rotary and oscillating instruments Type 1 plastic.

#### 1.1 Indications For Use

- The Enhance<sup>®</sup> Finishers are indicated for contouring and finalizing the form of composite and compomer restorations.
- The Enhance<sup>®</sup> PoGo<sup>®</sup> Polishers are indicated as the final step in polishing all properly contoured and finished composite resin and compomer restorations.

#### 1.2 Contraindications

- Do not reuse. Enhance<sup>®</sup> Finishers and Enhance<sup>®</sup> PoGo<sup>®</sup> Polishers components are single use, disposable items. Attempts to sterilize these items will adversely affect their performance.
- Enhance<sup>®</sup> Finishers and Enhance<sup>®</sup> PoGo<sup>®</sup> Polishers must not be used in High Speed (> 20,000 RPM) handpieces.

#### 1.3 Delivery forms (Some delivery forms may not be available in all countries)

- Enhance<sup>®</sup> Finishers are available in: Discs, Cups, Points
- Enhance<sup>®</sup> PoGo<sup>®</sup> Polishers are available in: Discs, Cups, Points

#### 1.4 Composition

**Enhance<sup>®</sup> Finishers:** Polymerized Urethane Dimethacrylate Resin; Aluminum Oxide; Silicon Dioxide

**Enhance<sup>®</sup> PoGo<sup>®</sup> Polishers:** Polymerized Urethane Dimethacrylate Resin; Fine Diamond Powder; Silicon Dioxide; Plastic Latch-type mandrel

#### 1.5 Compatible Substrates

Enhance<sup>®</sup> Finishers and Enhance<sup>®</sup> PoGo<sup>®</sup> Polishers are designed for use on self-cured and/or visible light cured composite, composite hybrid and compomer restoratives (see complete Instructions for Use of selected restorative material).

### 2. GENERAL SAFETY NOTES

Be aware of the following general safety notes and the special safety notes in other chapters of these Instructions for Use.

#### Safety alert symbol.

This is the safety alert symbol. It is used to alert you to potential personal injury hazards.

Obey all safety messages that follow this symbol to avoid possible injury.

#### 2.1 Warnings

- Enhance<sup>®</sup> Finishers and Enhance<sup>®</sup> PoGo<sup>®</sup> Polishers are designed for use without water. As with any rotary instrument, heat will build up with prolonged contact. Use at conventional speed (10,000-15,000 RPM) with light intermittent pressure. Excessive speed and/or pressure may result in heat buildup and may endanger the stability of the instruments. Heat buildup may cause pulp inflammation. Application of water-cooling during finishing and polishing assists with surface heat control required by some restorative materials (see complete Instructions for Use of restorative material). Application of an air-cooling stream during finishing and polishing assists with residue removal.
- The Enhance<sup>®</sup> Finishers and Enhance<sup>®</sup> PoGo<sup>®</sup> Polishers contain methacrylates which may be irritating to skin, eyes and oral mucosa and may cause irritant contact dermatitis in susceptible persons.
  - Avoid eye contact** to prevent irritation and possible corneal damage. In case of contact with eyes, rinse with plenty of water and seek medical attention.
  - Avoid skin contact** to prevent irritation. In case of contact, reddish rashes may be seen on the skin. If contact with skin occurs, remove material with cotton and alcohol and wash thoroughly with soap and water. In case of skin sensitization or rash, discontinue use and seek medical attention.
  - Avoid contact with oral soft tissues/mucosa** to prevent inflammation. If accidental contact occurs, remove material from the tissues. Flush mucosa with plenty of water and expectorate/evacuate the water. If inflammation of mucosa persists, seek medical attention.
- Use of Enhance<sup>®</sup> Finishers and Enhance<sup>®</sup> PoGo<sup>®</sup> Polishers components with high speeds and/or excessive lateral pressure may dislodge finisher from the mandrel which may be aspirated, ingested or cause soft tissue injury.
  - Aspiration:** Seek appropriate medical attention.
  - Ingestion:** Do not swallow or take internally. If accidental swallowing occurs, drink lots of water. If nausea or illness develop, seek medical attention immediately.
  - Eyes, skin, oral mucosa:** Thoroughly wash the affected area with water and seek appropriate medical attention.
- Care should be taken to have good control on any rotary instrument to protect patients from injury. Contact with soft tissues (skin, gingiva, mucosa) may result in injury to the tissue. If contact occurs, thoroughly wash the affected area with water and seek appropriate medical attention.

- Dental professionals and patients should wear protective eyewear while using the Enhance<sup>®</sup> Finishers and Enhance<sup>®</sup> PoGo<sup>®</sup> Polishers.
- Enhance<sup>®</sup> Finishers and Enhance<sup>®</sup> PoGo<sup>®</sup> Polishers should not be used with patients who have a known hypersensitivity to any of the components.

#### 2.2 Precautions

- This product is intended to be used only as specifically outlined in the Instructions For Use. Any use of this product inconsistent with the Instructions For Use is at the discretion and sole responsibility of the practitioner.
- Use with excess lateral pressure may dislodge Finisher or Polisher from mandrel (see Warnings) or cause quick deterioration and decreased effectiveness. This type of excess pressure may cause gouging of the restoration surfaces.
- Use in a well-ventilated area. Avoid inhaling airborne particulates.
- Protect adjacent restorations from accidental contact with rotating instruments.
- Devices marked "single use" on the labeling are intended for single use only. Discard after use. Do not reuse in other patients in order to prevent cross-contamination.
- Wear suitable protective eyewear, mask, clothing and gloves. Protective eyewear is recommended for patients.
- Use only clean and properly disinfected and/or sterilized handpieces. The hand piece must be in proper working condition.
- Enhance<sup>®</sup> Finishers and Enhance<sup>®</sup> PoGo<sup>®</sup> Polishers are provided ready for use and should not be processed. Use only new, clean unused parts, stored in covered storage, such as drawers and cabinets, away from potential contamination, and removed from their original packaging using either clean gloved hands or a clean suitable instrument at time of use. To reduce risk of infection, do not use Finishers or Polishers if packaging is damaged or compromised.
- Insert the instrument as deeply as possible without excessive use of force into the chuck and check to ensure that it is secure.
- Immediately remove any instruments that are damaged, bent or no longer run concentrically.
- Do not use the product if package is broken.
- Interactions:** None known.

#### 2.3 Adverse Reactions

- Product misuse may cause failure resulting in respiratory, digestive, eye, skin, or mucosa injury (see Contraindications, Warnings, and Precautions).
- Product misuse may cause excessive heat (see Contraindications and Warnings).
- Irritant contact dermatitis may occur in susceptible individuals (see Warnings).

#### 2.4 Storage conditions

- Inadequate storage conditions may shorten the shelf life and may lead to malfunction of the product.
- Store at temperatures between 10 °C/50 °F-27 °C/80 °F.
  - Use the product at room temperature.
  - Keep out of sunlight.
  - Do not freeze.
  - Protect from moisture. Dry storage is recommended.
  - Do not use after expiration date.
  - Do not use if the lot number and/or expiration date are missing or illegible.

### 3. STEP-BY-STEP INSTRUCTIONS

#### Handling

- Use only clean and properly disinfected and/or sterilized handpieces.
- Use only a clean, unused Enhance<sup>®</sup> Finisher, removed from the original, uncontaminated packaging.

- Place restoration or composite veneer as per standard techniques. Complete gross reduction of excess and general outline form of the restoration using Prisma<sup>®</sup> Finishing Burs, or diamond finishing instrument.

#### To reduce risk of excessive heat and/or instrument failure.

- Use only conventional speed (10,000-15,000 rpm) handpieces.
- Use moderate to light, intermittent pressure.
- Water-cooling assists with surface heat control.

- Insert an Enhance<sup>®</sup> Finisher into a conventional speed contra-angled handpiece and continue finishing. **NOTE:** Enhance<sup>®</sup> Finishers and Enhance<sup>®</sup> PoGo<sup>®</sup> Polishers MUST NOT BE USED IN HIGH SPEED HANDPIECES. The aggressiveness of the Enhance<sup>®</sup> Finishers is controlled by the pressure applied to the surface of the composite. Use moderate to light, intermittent pressure in a buffing motion. Frequently raise the finisher from surface contact.

- Once final contours are achieved, proceed to polishing with Enhance<sup>®</sup> PoGo<sup>®</sup> Polishers as outlined below, or Prisma<sup>®</sup> Gloss<sup>™</sup> Composite Polishing Pastes per manufacturer's instructions for final high surface luster.

#### TECHNIQUE HINTS

- Enhance<sup>®</sup> Finishers and Enhance<sup>®</sup> PoGo<sup>®</sup> Polishers are designed to be used without water. If desired or directed by restorative material manufacturer's Instructions for Use, water-cooling may be applied without affecting performance.
- The aggressiveness of the Enhance<sup>®</sup> Finishers is controlled by the pressure applied to the surface of the composite. The greater the pressure, the more material is removed; lighter pressure leaves a smooth surface without removing bulk.
- Prior to proceeding to polishing with Enhance<sup>®</sup> PoGo<sup>®</sup> Polishers or Prisma<sup>®</sup> Gloss<sup>™</sup> Composite Polishing Pastes, the restoration should be finished to a final contour. Surface should be smooth and defect free.

For best results in larger restorations, the use of an intermediate finishing instrument such as an Enhance<sup>®</sup> Finisher in a conventional contra-angled handpiece will aid in removing scratches and establishing the desired smooth, defect free surface.

- Insert an Enhance<sup>®</sup> PoGo<sup>®</sup> Polisher into a conventional speed contra-angled handpiece. **NOTE:** Enhance<sup>®</sup> Finishers and Enhance<sup>®</sup> PoGo<sup>®</sup> Polishers MUST NOT BE USED IN HIGH SPEED HANDPIECES.

#### Handling

- Use only clean and properly disinfected and/or sterilized handpieces.
- Use only a clean, unused Enhance<sup>®</sup> PoGo<sup>®</sup> Polisher, removed from the original, uncontaminated packaging.

#### To reduce risk of excessive heat and/or instrument failure.

- Use only conventional speed (10,000-15,000 rpm) handpieces.
- Use moderate to light, intermittent pressure.
- Water-cooling assists with surface heat control.

- Begin polishing by applying light intermittent pressure at moderate speed to the restoration surface. When using the disc, the flat broad surface of the Enhance<sup>®</sup> PoGo<sup>®</sup> Polisher should be used in contact with the restoration to optimize its effectiveness. To increase surface luster, decrease the pressure applied to the restoration surface using a light buffing motion. **Technique Tip:** The flexibility of Enhance<sup>®</sup> PoGo<sup>®</sup> Polishers, when used lightly, allows polishing of established line angles without alteration of form.

Shape.....	Recommended Application
Enhance® Disc.....	Facial/Smooth surfaces
Enhance® PoGo® Disc.....	Facial surfaces and established line angles
Enhance® & Enhance® PoGo® Cup.....	Cervical and proximal line angle areas
Enhance® & Enhance® PoGo® Point .....	Posterior occlusal and concave lingual surfaces

#### 4. HYGIENE, PROCESSING AND DISPOSAL

##### Cross-contamination - To reduce risk of Infection.

- Do not reuse single use Enhance® Finishers and Enhance® PoGo® Polishers. Dispose of in accordance with local regulations.
- Packing cannot be reprocessed. Dispose of contaminated packaging in accordance with local regulations.
- During treatment clinicians with patient contact should only handle items to be used.
- Packaging should be handled with clean/disinfected gloves, outside of the patient treatment operator.
- Liquid contact may damage package label.
- When not in use, all supplies and instruments should be in covered storage, such as drawers and cabinets, and away from potential contamination.
- Dispensing materials with clean/disinfected gloves or hands or suitable transfer forceps in a separate room, bringing only what will be used into the operator is strongly recommended.

CAUTION

##### 4.1 Disposal

- Based on the information in the SDS, the waste generated by the disposal of the product does not present characteristics of physical danger, for the person or the environment; it can therefore be managed, according to local regulations, as a non-hazardous waste.
- Contaminated instruments that have been in contact with the patient must be disposed of as hazardous waste at risk of biological contamination.
- Always wear gloves when handling packaging and instruments.

#### 5. LOT NUMBER, EXPIRATION DATE AND CORRESPONDENCE

1. Do not use after expiration date. Do not use if the lot number and/or expiration date are missing or illegible. ISO standard uses: "YYYY-MM-DD".
2. The following numbers should be quoted in all correspondences:
  - Reorder number
  - Lot number on package
  - Expiration date on package
3. Any serious incident in relation to the product should be reported to the manufacturer and the competent authority according to local regulations.
4. Device Identification: Basic UDI-DI ++DOOFINISHPOLISHMATSNV

© 2024 Dentsply Sirona Inc. All Rights Reserved. **562031WEB** (R 11/22/24)

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